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INSTRUCTIONS FOR THE USE OF COVER SCREWS

1. Product description

A cover screw is a titanium item that may be comprised of one or more parts that are threaded for screwing to an implant or transepithelial element.

It is important to note that the terms *cover screw* and *cover cap* are equivalent.

2. Planned use:

The purpose of the device is to protect the connection of the element onto which it is screwed (implant or transepithelial element) from invasion by the patient's soft tissues or other substances.

3. Contraindications

Allergy and hypersensitivity to the chemical components of the materials used (medical grade titanium).

4. Side effects and interactions

Follow the instructions for use recommended by the manufacturer. Incorrect use of the product may cause injury to the patient.

5. Cleaning, disinfection and sterilisation

Cover screws are sold unsterilised. Before using them, the healthcare professional must perform cleaning and disinfection before sterilisation.

Bioner SA always recommends performing previous cleaning with products commonly used in dentistry, with emphasis on ultrasonic baths or automatic cleaning and disinfection methods before proceeding with sterilisation.

Remove the product from its original packaging and place it in a suitable container before sterilisation.

Sterilisation should be performed in sterilisation bags or boxes in a wet steam autoclave at 121°C for 15 minutes or 134°C for at least 3 minutes, as per UNE-EN-ISO 17665-1:2007.

Do not use the product after sterilisation if there is any detectable deterioration of the container.

Cover screws, as well as all accessories produced by Bioner SA, should not be reused; they are products for single use only.

6. Warnings

Before use, make sure that the material is in perfect condition and there are no loose parts to prevent the inhalation or swallowing of one or more components. Furthermore, it is highly recommended to make sure that the product is clean and disinfected and the part in contact with the patient is clean and free of any residue.

The healthcare professional using the product must have experience in using it safely and in accordance with instructions for use provided by BIONER SA, the said professional being liable for the suitability of the component to the patient's needs.

7. Precautions

It is especially important to ensure that each reference is used in combination with dimensionally compatible elements as indicated on the product label.

8. Compatibility

BIONER cover screw family references must be used in conjunction with compatible components. Carefully examine the components available in the company product catalogue for suitability before insertion.

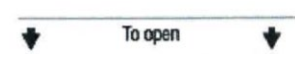
The dimensions ensure compatibility with the components matching each specific reference. The compatibility of each reference is guaranteed by BIONER SA in-process controls.

9. Validity

These instructions for use correspond to the current version of references for the cover screw family as indicated at the foot of the document.

10. Storage

Elements of this family must be preserved and stored in non-extreme environmental conditions, avoiding prolonged exposure to sunlight.

ISO 15223 Symbol	Description
	Indicates the direction for opening the blister
	BIONER SA logotype
REF	Catalogue reference
LOT AOFAAXXXX	Batch code
 DD/MM/YY	Date of Manufacture
	Refer to the instructions for use
	Do not use if package is damaged
Family	Product family name / additional info
Mat = Ti	General indication of the material used
q=1	Indicates the number contained in each blister
	Non-sterile
	QR code (reference and batch)
	Do not reuse

	Precautions
	Manufacturer
	EC Marking

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